

## MEDICATION CUP CLEANING INSTRUCTIONS



1 Use a clean dish and dishwashing liquid



2 Add a few drops of dishwashing liquid



3 Use clean warm water



4 **IMPORTANT**  
**Add the Protective Cap**



5 Rinse the medication cup thoroughly

### **IMPORTANT** FINAL STEP

Rinse both sides with **DISTILLED WATER**

**Distilled Water can be purchased at any Supermarket**



### **Spare Medication Cups:**

It is highly recommended to have spare medication cups on hand to ensure continuity of treatment. Spare medication cups may be purchased by calling 0417 415 355.